


TNT Medpak Thermo Packing Guide UN3373 Biological Substance

Step 1 – Dispose of the clear outer wrapper



Peel & Stick Top Section of label To the top of the box

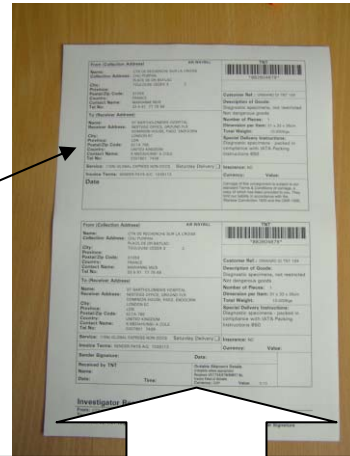

Step 2 – Carefully open and remove the lid. Remove any transit documentation from inside the lid.



Step 6 – Sign and date any consignment note information. For non EU countries a Commercial Invoice will also require completion

Step 3 – Remove the Biobottle carefully from the Dry Ice. Unscrew the top and remove the inner protection
Please note gloves should be worn.



Step 5 – Carefully place the Biobottle into the Dry Ice and replace the lid. Close up the box ready for shipping

Step 4 – Place the specimen phial into the protection bag and place this inside the Biobottle

